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****EN**^②**
DIGITAL HORSE THERMOMETER for measuring the temperature of horses (for private use only)

Distributor & contact details

Waldhausen GmbH & Co KG
Von-Hünefeld-Straße 53 | 50829 Cologne | Germany
info@waldhausen.com, www.waldhausen.com

****ES**^③**
Scope
Thermometer,
1.5 V DC battery, alkaline manganese cell, 25-40 mAh, LR41/L736F, AG3

Preparation
Take the thermometer out of the packaging. Disinfect the measuring tip with a suitable disinfectant. Switch on the device using the **on/off button**. Wait until the display shows **‘Lo’** or **‘Ready’**.

Operation
Carefully insert the measuring tip rectally into the horse (approx. 5–7 cm deep). Hold the thermometer steady until a beep sounds. Remove the thermometer and read the temperature on the display.

After measurement
Switch off the device. Clean and disinfect the tip again. Store the thermometer in a dry place out of the reach of children.

Cleaning
Use an alcoholic disinfectant or warm water with mild soap. Do not immerse in water or put in the dishwasher. Make sure that no liquid enters the battery compartment or the display. Allow to dry completely after cleaning.

Safety instructions & disposal
Only suitable for rectal use in horses. Do not use on other animals or humans. Clean thoroughly before and after each use.
Do not use if the housing is damaged, the tip is scratched or the display is faulty.
Batteries must not be thrown into the fire or household waste – return to collection points. Keep out of reach of children – contains small parts that may be swallowed. Do not use in heavy rain or in damp environments.
Do not open the device.

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