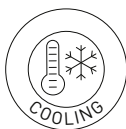
WALDHAUSEN

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E Instructions for Waldhausen Health + Care Horse Boots

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E

Instructions for use for Waldhausen Health + Care Horse Boots.....	11
Delivery contents.....	11
Instructions to pay attention to before use.....	11
What features does the Waldhausen Health + Care Horse Boot have?.....	12
Cool + Vibrate.....	13
Heat + Vibrate.....	13
How do I attach the Waldhausen Health + Care Horse Boot?.....	14
Use and duration of use.....	16
Important safety Instructions.....	16
Important care instructions.....	16
Disposal instructions.....	16

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INSTRUCTIONS FOR USE FOR WALDHAUSEN HEALTH + CARE HORSE BOOTS

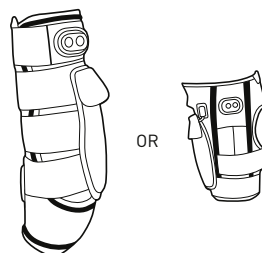
In the event of injuries, to warm up and loosen up, or after strenuous sport, the new Waldhausen Health + Care Horse Boot by Waldhausen supports regeneration of horse's legs by means of 3 features – Vibrate, Cool and Heat. The Waldhausen Health + Care Horse Boot has various different areas of application. The horse boot cools the leg after exercise and thus promotes the rehabilitation phase. The Waldhausen Health + Care Horse Boot can also be used for support after and during acute and/or chronic injuries. When used in this way, it can accelerate the healing process and prevent/reduce swelling. The Waldhausen Health + Care Horse Boot can both cool legs down and warm them up thanks to the Heat and Cool features. This can lead to rapid regeneration and prevention of injuries.

INSTRUCTIONS TO PAY ATTENTION TO BEFORE USE

Please charge the power bank before use. Then put this in the side compartment of the case provided for the power bank. Please make sure that the power bank and the cooling pack are stored separately from each other. Plug the supplied USB connection cable into the power bank in order to later connect it to the horse boot

If you want to use the cool packs, put the cool packs in the fridge/freezer for several hours before use. The cool packs can then be placed in the compartments provided on the inside of the horse boot. We recommend transporting the horse boot in the Waldhausen Waldhausen Health + Care Cool Bag. This way, the cool packs are also kept cool during transport.

Delivery contents:

OR

- 1x** WALDHAUSEN HEALTH + CARE HORSE BOOT WITH INTEGRATED HEATING AND VIBRATION ELEMENTS

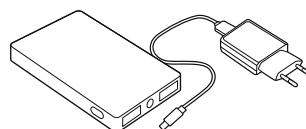
- 2x** COOL PACKS (HORSE ANKLE BOOT)

- 4x** COOL PACKS (HORSE BOOT)

- 1x** USB CONNECTION CABLE

- 1x** POWERBANK (5V/2A), INCLUDING CASE, EU ADAPTER CHARGING CABLE WITH MICRO USB

- 1x** INSULATING BAG

WHAT FEATURES DOES THE WALDHAUSEN HEALTH + CARE HORSE BOOT HAVE?

The Waldhausen Health+Care Horse Boot has different features. These include: Cool, Heat and Vibrate.

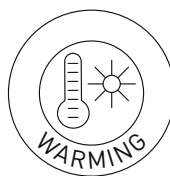
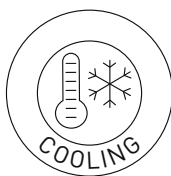
All features can be used independently of one another.

¹W-HEALTH AND CARE HORSE BOOT

²WALDHAUSEN HEALTH + CARE HORSE ANKLE BOOT

Cool + Vibrate

After high stress or in the case of swelling or injuries to the tendons, the Waldhausen Health+Care Hose Boot can actively support the healing process by means of the two Cool & Vibrate features. The vibrations help realign the tendon fibres and activate blood circulation in the horse's leg. The Vibrate feature can be regulated across 3 levels for this purpose.

The flexible cool packs minimise swelling and have a pain-relieving and anti-inflammatory effect. Of course, the horse boot can also be used without the cool pack.

Level 1 (red)
Constant
vibration

Level 2 (white)
Pulsating
vibration

Level 3 (blue)
Slow-pulsating
vibration

Heat + Vibrate

The heating elements are made of carbon fibres. With the help of the special far-infrared technology, the horse boot's warming function can improve blood circulation and release tension in muscles after high amounts of stress, as well as warm the leg up. Especially in winter, the Waldhausen Health+Care Horse Boots can be used for warming up the horse's leg and stimulating the blood circulation within it during longer periods of rest in the box. This causes the tendons to become more elastic and can reduce the risk of injury. The Vibrate and Heat features of the horse boot can be regulated using three different levels, and can thus be optimally adapted to the needs of your horse.

Level 1 (red)
50°

Level 2 (white)
45°

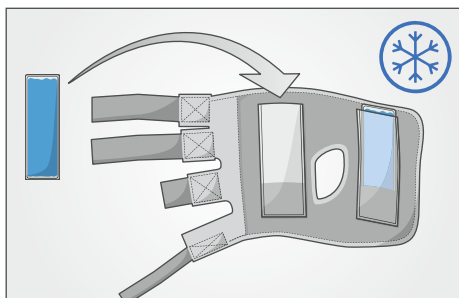
Level 3 (blue)
40°


Battery power with simultaneous use of the vibration and heat function approx. 90 minutes. Battery power may vary at extreme temperatures.

HOW DO I ATTACH THE WALDHAUSEN HEALTH + CARE HORSE BOOT?

1 First place the cool packs in the compartments provided in the horse boot. (This step is only necessary if you want to use the horse boot to cool the horse's legs. When using them to warm their legs, no cool packs are placed in the compartments.)


2 Attach the horse boot to the horse's leg using the Velcro fasteners provided. Make sure that the horse boot cannot slip.

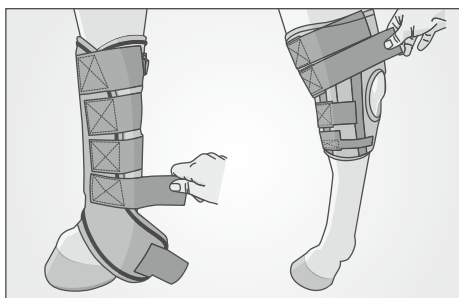
3 Stow the power bank in the pocket provided and plug in the USB cable.


Velcro-fasten the case with the power bank to the horse boot. It is important that it sits firmly on the horse boot so that it does not slip or come loose.

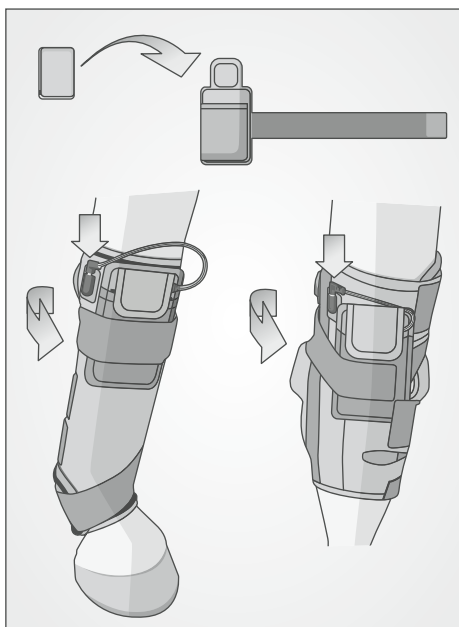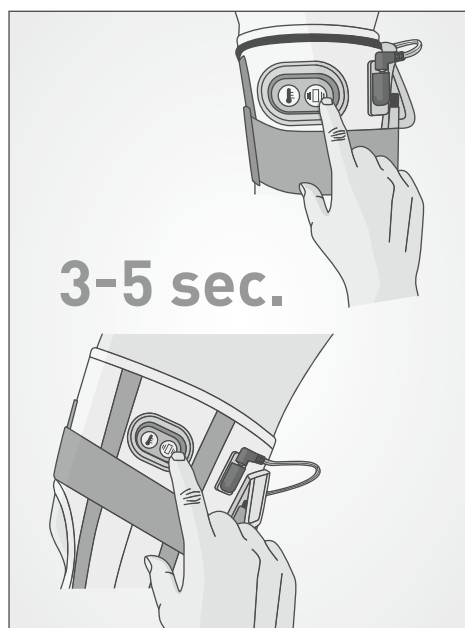Then connect the power bank cable to the Waldhausen Health + Care Horse Boot.

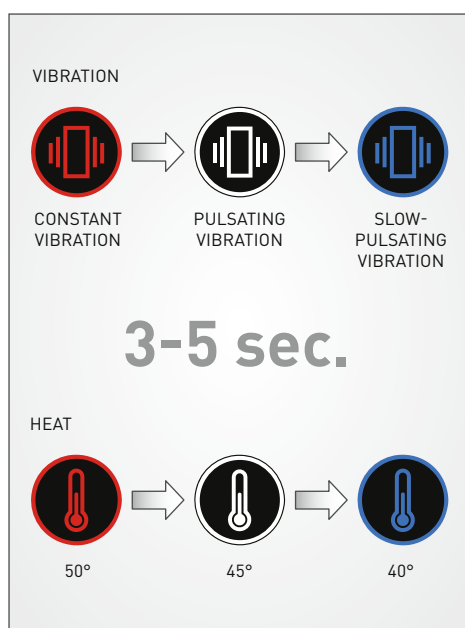4 After the horse boot has been supplied with power, the horse boot is switched on using the Vibrate or Heat button. Please select the button of the desired function. To do so, the button must be pressed for 3–5 seconds. The horse boot will then switch on at the first level.

5 To change the level, again, press the button for 3–5 seconds, then the horse boot will switch to the next level.

To switch the horse boot off again, press the button of the feature being used (Vibrate or Heat button) for about 3 to 5 seconds.

USE AND DURATION OF USE

Depending on the intensity and vibration level, we recommend using the horse boot for between 5 and a maximum of 15 minutes per horse's leg. When using the horse boot for the first time, we recommend that you only use the boot at level 1–2 to allow the horse and horse's leg to get used to the vibrations. If you are unsure about how to use the horse boot or about the intensity and duration of use, please clarify this in advance with your veterinarian or physiotherapist.

The Waldhausen Health+Care Horse Boot may only be used under supervision. Please only use the horse boot in accordance with the instructions for use and observe the following safety instructions and the care instructions:

Important safety Instructions

- Only use under supervision
- Use only at rest, not when in motion
- Make sure that your horse does not „nibble at“ the horse boot while it is in use
- Make sure that the cable is properly stowed and does not droop down during use, so as to avoid the risk of injury from the cable
- Do not use on open wounds
- Please clarify the intensity and duration of use with your veterinarian or physiotherapist
- Please check the temperature on the horse's leg regularly during use
- Do not use in case of infection
- Do not use in the event of fractures

- Do not place things on the horse boot of exert heavy pressure on it, as it contains sensitive electronic elements
- Never heat up cool packs
- Only for horses

The Waldhausen Health+Care Horse Boots were developed to support horses after high levels of stress or injuries during the rehabilitation phase and to minimise swelling. The horse boots are not a medical device.

Important care instructions

- Do not bring into contact with moisture
- Due to electronics in the horse boot, please do not wash. The horse boot can be cleaned with a brush
- Do not bend, wring out or allow to drop
- Store in a dry place and do not expose to extreme temperatures or other environmental influences
- Do not use damaged batteries/power banks

Disposal instructions

All electronic devices and batteries must be disposed of separately. Do not throw these devices into household waste, but ensure they are disposed of properly.

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WALDHAUSEN GmbH & Co. KG