
WALDHAUSEN

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
- DE** Gebrauchsanweisung Waldhausen Health + Care Infrared FlexPad
- GB** Instructions for use for Waldhausen Health + Care Infrared FlexPad
- FR** Mode d'emploi des Waldhausen Health + Care Infrared FlexPad
- IT** Istruzioni per l'uso Waldhausen Health + Care Infrared FlexPad
- ES** Instrucciones de uso Waldhausen Health + Care Infrared FlexPad
- NL** Gebruiksaanwijzing Waldhausen Health + Care Infrared FlexPad
- DK** Brugsanvisning Waldhausen Health + Care Infrared FlexPad
- FI** Käyttöohje Waldhausen Health + Care Infrared FlexPad
- SE** Bruksanvisning Waldhausen Health + Care Infrared FlexPad
- NO** Bruksanvisning Waldhausen Health + Care Infrared FlexPad
- PL** Instrukcja obsługi urządzenia do masażu Waldhausen Health + Care Infrared FlexPad
- BG** Инструкция за употреба Waldhausen Health + Care Infrared FlexPad
- CZ** Instrukce k použití Waldhausen Health + Care Infrared FlexPad
- EE** Kasutusjuhend Waldhausen Health + Care Infrared FlexPad
- GR** Οδηγίες Χρήσης Συσκευής Μασάζ Waldhausen Health + Care Infrared FlexPad
- HR** Upute za uporabu Waldhausen Health + Care Infrared FlexPad
- HU** Használati útmutató Waldhausen Health + Care Infrared FlexPad
- LT** Gamybos instrukcija Waldhausen Health + Care Infrared FlexPad
- LV** Lietošanas instrukcija Waldhausen Health + Care Infrared FlexPad
- MT** Istruzzjonijiet ta' Użu ġenerali għal Waldhausen Health + Care Infrared FlexPad
- PT** Manual de Instruções Waldhausen Health + Care Infrared FlexPad
- RU** Инструкция по применению Waldhausen Health + Care Infrared FlexPad
- SI** Navodila za uporabo Waldhausen Health + Care Infrared FlexPad
- SK** Návod na použitie Waldhausen Health + Care Infrared FlexPad



Art.-Nr./Product no./N° Art.:
388001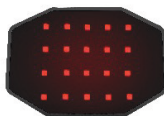

www.waldhausen.com

Waldhausen GmbH & Co.KG
Von-Hünefeld-Str. 53 | 50829 Köln | Germany
info@waldhausen.com

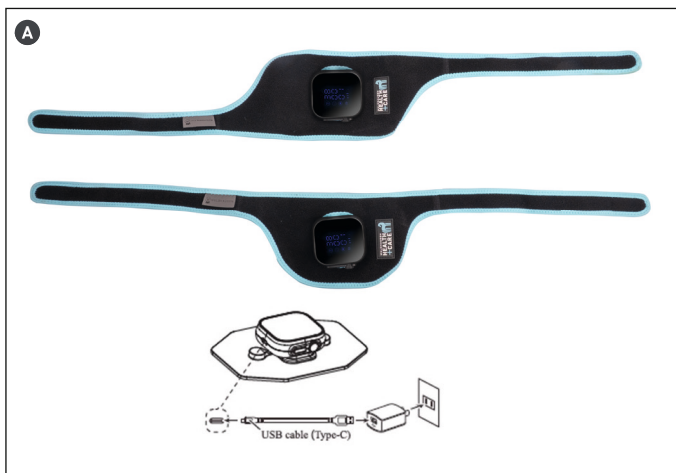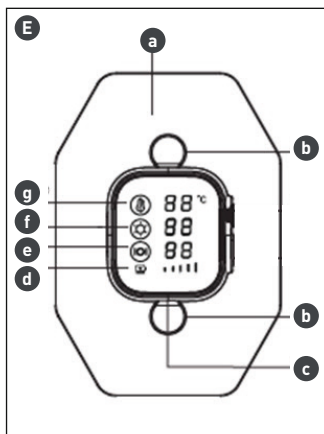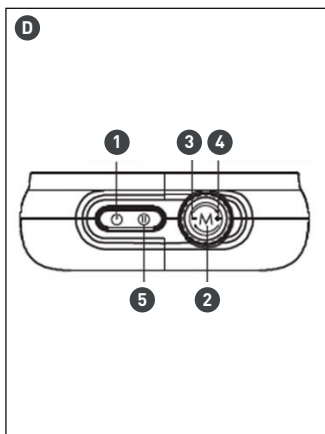Hersteller

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GB Instructions for use for
Waldhausen Health + Care Infrared FlexPad

GB

Distributor

Waldhausen GmbH & Co. KG, Von-Hünefeld-Str. 53, 50829 Cologne, Germany
info@waldhausen.com, www.waldhausen.com

1. Scope of delivery

Infrared FlexPad – Main Device

USB-C cable

Instruction manual

Two fastening straps (1x leg and 1x neck)

2. Technical details

Model: Waldhausen Health + Care Infrared FlexPad – HXR-A090
Dimensions: 800 x 110 x 35 mm, 161 g
3,7V, 1900 mAh, 18 W, 18,43 MHz
Lithium-ion battery, USB-C
Standards: CE Protection class: I
Infrared light intensity: 3 levels Spectrum: 660nm + 850nm, irradiance Level 1: 11 mW/cm ² , Level 2: 18.5 mW/cm ² , Level 3: 32 mW/cm ²
Temperature: 38 °C–48 °C ± 3 °C
Vibration: 3 levels: Level 1: Light continuous vibration, Level 2: Continuous vibration, Level 3: Intervals
Runtime: 1.5-2 hours based on different function levels (heat, infrared, vibration)
Charging time: ~2-3 hours

3. Applications

The Infrared FlexPad for horses is an innovative solution for promoting muscle relaxation, blood circulation, and regeneration. The targeted deep heat gently warms the horse's muscles, relieving tension and stimulating the metabolism. The pad is particularly suitable for use before training, for relaxation after exercise, or as an accompanying therapy for muscular problems.

Typical applications for the Infrared FlexPad on horses: The infrared pad is versatile and particularly suitable for supportive use in the following areas:

Muscle tension and muscle regeneration: The heat can stimulate blood circulation in the muscles and thus contribute to relaxation before or after intensive training sessions. It is also often found to be beneficial in cases of muscular overload or hardening.

Joint support: The effect of heat can help loosen the joints and promote mobility. The pad is also often used to provide support for age-related mobility restrictions.

Tendon and ligament areas: In subacute phases after strain or minor injuries, heat can help reduce swelling and improve blood circulation. Caution: Do not use in acute phases, as heat can be counterproductive in such cases.

Preparation before training: The pad is suitable for activating the muscles before training. Its use can help to gently prepare the muscles and reduce the risk of tension.

Instructions for use:

Never use in cases of acute inflammation (e.g., fresh tendonitis, open wounds). Consultation with a veterinarian or equine therapist is recommended, especially for chronic or complex problems.

4. Commissioning – (Fig. A + B)

Preparation: Charge the device completely while it is switched off; charging time approx. 2–3 hours. Please use the enclosed USB-C cable with a suitable adapter (Fig. A). Ensure that the horse is clean and dry. The pad should also be dry and undamaged. Attach the Infrared FlexPad to the corresponding fastening strap for the leg or neck. The Infrared FlexPad can be easily inserted from below through the opening provided (Fig. B).

Placement (Fig. C): Once attached, carefully place the Infrared FlexPad on the desired area of the horse's body – leg or neck. Make sure it fits well and secure it with the Velcro fasteners (leg: one Velcro fastener at the top, one at the bottom; neck: one Velcro fastener through the eyelet of the halter and one Velcro fastener at the top).

Switching on (Fig. D):

- 1) On/Off
- 2) Function selector
- 3) Counterclockwise rotation -
- 4) Clockwise rotation +
- 5) Pause/Play

Switch on the Infrared FlexPad according to the instructions and select the desired heat/infrared &/vibration level. Start with a low intensity, especially when using for the first time. Press and hold the on switch: the device switches to standby mode and automatically switches off after 60 seconds of inactivity. Select the function by pressing the round button function selector. Select the temperature display, turn the knob clockwise to increase the temperature and counterclockwise to decrease

the temperature. Select the desired temperature, 38-48 °C, +/- 3 °C temperature fluctuations due to environmental conditions. Press the function selector again to select the light intensity – 3 levels. Press the function selector again to select the vibration intensity – 3 levels. The device switches off automatically after 15 minutes.

Duration of use: The recommended treatment duration is approx. 15 minutes. Monitor the horse's behavior during this time and stop the treatment if the horse becomes restless or uncomfortable.

After use: Carefully remove the pad and allow the horse to rest for a few minutes before moving or working. Store the Infrared FlexPad in a safe, dry place away from direct sunlight.

5. Functions (Fig. E)

- a) Infrared massage heat pad
- b) Massage motor
- c) USB C port
- d) Charging status
- e) Vibration/massage indicator
- f) Infrared indicator
- g) Temperature indicator

6. General instructions for use

- The device is intended exclusively for use on horses.
- It is only suitable for private use and not for medical or commercial purposes.
- The use of the device should be comfortable at all times.
- If this is not the case, please stop using it immediately.
- Only use the device when the horse is awake and only use one device at a time.
- Please read the instructions for use before using the device for the first time.
- Please check that the delivery is complete and undamaged. Only use and clean the device when it is undamaged and dry, and in a dry environment. The device is only intended for users aged 8 years and above. Users with physical, sensory, or mental impairments may only use the device under supervision.
- Do not use the device for longer than 15 minutes; allow it to cool down before using it again.
- Improper operation and use of this device can result in injury and damage.
- By using this device, you are deemed to have understood, acknowledged, and accepted all the terms and conditions of this document. The manufacturer is not liable for damage resulting from improper use.

**7. Safety instructions**

Before use, please check the device for damage.

Do not use the device while it is charging. Only charge and use the device in dry conditions and environments. No water must be allowed to enter the device.

The device must not be modified, repaired, or opened.

Keep the device out of the reach of children and vulnerable groups.

Protect the device from extreme temperatures, direct sunlight, fire, heat, magnets, and do not use the device under covers. Ensure that the device is clean and that no hair is caught in it. Avoid contact with pointed and sharp objects, acids, and flammable liquids.

Do not use any other attachments.

Charge fully to ensure optimal performance.

Caution: if the device smells unusual, becomes hot, or exhibits other abnormalities, stop using it immediately.

8. Storage and disposal

Caution – do not dispose of the device in household waste; disposal must be carried out in accordance with local regulations.

The device should be stored at room temperature and not exposed to damp, very high or low temperatures.

Clean the device with a damp cloth, not with cleaning solutions or toxic chemicals.

9. Warranty

The manufacturer provides a 2-year warranty from the date of invoice for countries within the European Union. Statutory warranty rights remain unaffected.

Exclusion of warranty claims: Damage caused by normal wear and tear, environmental influences, or improper use. Failure to observe safety instructions.

In the event of a warranty claim, please contact your dealer with proof of purchase. Waldhausen GmbH & Co KG hereby declares that the Waldhausen Health + Care Infrared FlexPad complies with current EU directives.

The complete EU declaration of conformity can be found at www.waldhausen.com/Service/.

10. Troubleshooting

The device does not switch on	Please check whether the device is charged. Press and hold the power button.
The device switches off automatically	The device switches off automatically after 15 minutes. If it switches off automatically earlier, check whether the battery is sufficiently charged.

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Waldhausen Health + Care Infrared FlexPad

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Waldhausen GmbH & Co. KG, Von-Hünefeld-Str. 53, 50829 Cologne, Allemagne
info@waldhausen.com, www.waldhausen.com

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WALDHAUSEN GmbH & Co. KG