
WALDHAUSEN

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- DE** Gebrauchsanweisung Waldhausen Health + Care TecSupreme Gamasche
- GB** Instructions for use for Waldhausen Health + Care TecSupreme Boot
- FR** Mode d'emploi des Waldhausen Health + Care TecSupreme guêtre
- IT** Istruzioni per l'uso Waldhausen Health + Care TecSupreme gambale
- ES** Instrucciones de uso Waldhausen Health + Care TecSupreme polaina
- NL** Gebruiksaanwijzing Waldhausen Health + Care TecSupreme gamasche
- DK** Brugsanvisning Waldhausen Health + Care TecSupreme Gamasche
- FI** Käyttöohje Waldhausen Health + Care TecSupreme säärystin
- SE** Bruksanvisning Waldhausen Health + Care TecSupreme Gamasche
- NO** Bruksanvisning Waldhausen Health + Care TecSupreme Gamasche
- PL** Instrukcja obsługi urządzenia do masażu Waldhausen Health + Care TecSupreme getry
- BG** Инструкция за употреба Waldhausen Health + Care TecSupreme гамаша
- CZ** Instrukce k použití Waldhausen Health + Care TecSupreme gamasche
- EE** Kasutusjuhend Waldhausen Health + Care TecSupreme säärekaitsed
- GR** Οδηγίες Χρήσης Συσκευής Μασάζ Waldhausen Health + Care TecSupreme γκέτα
- HR** Upute za uporabu Waldhausen Health + Care TecSupreme gaiter
- HU** Használati útmutató Waldhausen Health + Care TecSupreme gamasche
- LT** Gamybos instrukcija Waldhausen Health + Care TecSupreme Gamasche
- LV** Lietošanas instrukcija Waldhausen Health + Care TecSupreme Gamasche
- MT** Istruzzjonijiet ta' Użu generali għal Waldhausen Health + Care TecSupreme gaiter
- PT** Manual de Instruções Waldhausen Health + Care TecSupreme polaina
- RU** Инструкция по применению Waldhausen Health + Care TecSupreme гетры
- SI** Navodila za uporabo Waldhausen Health + Care TecSupreme gamasche
- SK** Návod na použitie Waldhausen Health + Care TecSupreme gamasche

Art.-Nr./Product no./N° Art.:
389601

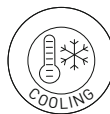WALDHAUSEN
**HEALTH
+ CARE** 

www.waldhausen.com

Waldhausen GmbH & Co. KG
Von-Hünefeld-Str. 53 | 50829 Köln | Germany
info@waldhausen.com

1

2

3

4

5

Inverkehrbringer

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Scope of delivery

1 x Waldhausen Health + Care TecSupreme gaiter
 4 x cooling packs (6 x 18 cm)
 1 x USB C charging cable 5V 2A
 1 x insulated storage bag
 Instructions for use

Technical details

Model: Waldhausen Health + Care TecSupreme gaiter		
Dimensions: 34 x 13 x 12 cm		
Weight: 420 g		
4000mAh/3.7V Input: USB-C 5V-2A; 8W		
Heat settings 45/55/65 °C	3 massage levels Red: nonstop White: short interval Blue: long interval	Duration approx. 90 minutes, depending on the heating and vibration intensity No cooling during the heating or massage application.
Reorder cooling packs: 336697 2 pieces		

1. General instructions for use

- Please read the instructions for use before using the product for the first time.
- Please check that the delivery is complete and undamaged. Only use and clean the gaiter when it is undamaged and dry, and in a dry environment.
- Improper handling and use of this gaiter can lead to injury and damage. By using these gaiters, you are deemed to have understood, acknowledged, and accepted all the terms and conditions contained in this document.
- For private use only; not for medical or commercial purposes.
- The gaiters are intended for users aged 8 years and older. Users with physical, sensory, or mental impairments may only use the device under supervision.
- The gaiter is intended exclusively for use on horses that are awake.
- The use of the gaiter should be comfortable for the animal at all times; in all other cases, please discontinue use immediately.
- The gaiter should always be used in consultation with a veterinarian, especially if the horse has serious health problems.
- The manufacturer is not liable for damage caused by improper use.

2. Applications

The TecSupreme gaiter is primarily used to promote blood circulation, relax muscles, and support regeneration. The gaiter combines heat and massage effects, which can help to loosen muscles and promote flexibility. They can be used before training to warm up the muscles and prepare them for exertion, as well as after training to relax and support regeneration.

The application can also be pleasant for muscular tension or after prolonged periods of rest and can improve mobility. The heat supports blood circulation and metabolism in the tissue, while the massage can create pleasant stimulation. The gaiter can also be used during boxing rest or after periods of rest to maintain blood circulation and prevent muscle breakdown.

Before training: To prepare and warm up the muscles before exercise.

After training: To support muscle relaxation and regeneration.

For muscle tension: Can have a soothing effect and promote mobility.

During or after rest periods/box rest: To support blood circulation and active muscle maintenance.

Wellness & care: Also suitable as a relaxation and wellness treatment for horses without any specific reason.

3. Application / Commissioning

Figure 1 View of gaiter with control panel

- a) Fastener
- b) Control panel

Figure 2 View of gaiter without control panel

Figure 3 Control panel when switched on

- a) On/Off
- b) Heat setting
- c) Vibration setting
- d) USB C connection
- e) Charge level indicator

Figure 4 Cooling packs (inside view)

Figure 5 Storage bag

Start-up

Fully charge the TecSupreme gaiter; charging time approx. 2 hours. Please use the enclosed USB-C cable with a suitable adapter for charging. The indicators flash during charging. When the device is fully charged, all indicators will light up. Please disconnect the fully charged device from the charging unit immediately.

Cool the cooling packs in a refrigerator/freezer for several hours before use. The cooling packs can then be inserted into the compartments provided on the inside of the gaiter. We recommend using the included insulated bag for transport.

Heat and vibration function:

The intensity levels can be conveniently changed by pressing repeatedly.

To turn off, press the buttons on the control panel for 3 seconds.

3 vibration levels

	Level 1 (red): Constant vibration
	Level 2 (white): Pulsating vibration
	Level 3 (blue): Slow pulsating vibration

3 heat settings

	Level 1 (red) 65 °C
	Level 2 (white) 55 °C
	Level 3 (blue) 45 °C

The heating and massage functions can be used simultaneously, but not the cooling function. Please familiarize yourself with the product and its functions, ideally not on the horse. Ideally, you should clean the horse before use. When using the product on a horse for the first time, it is recommended that a second person hold the horse in place rather than tying it up. Apply the gaiter slowly and carefully, starting with the heat function at the lowest setting. Once the horse has become accustomed to this, add the massage function at the lowest setting. Pay attention to the horse's reaction and, if in doubt, remove the gaiter immediately. Allow the horse to become accustomed to the product gradually.

4. Safety instructions

- Before use, please check the gaiter, cooling packs, and control panel for damage.
- Do not use the gaiter while it is charging.
- The gaiter must not be modified, repaired, or opened. Please contact your dealer or the manufacturer.
- Keep/use the gaiter out of the reach of children and vulnerable groups.
- Protect the gaiter from extreme temperatures, direct sunlight, fire, heat, magnets, and moisture. Do not use the device under pillows or covers.
- Ensure that the moving parts are free and that no hair is caught in them.
- Avoid contact with pointed and sharp objects, acids, and flammable liquids.
- Do not use any other attachments or cables.
- Charge fully to ensure optimal performance.
- Caution: if the device smells unusual, becomes hot, or shows other abnormalities, stop using it immediately.
- Do not leave the horse unattended during use.
- Use only when stationary, not in motion.

5. Storage, cleaning, disposal

- After use, clean the gaiter and cooling packs with a damp cloth.
- Store them in the bag in a dry place.
- Please do not wash, dry clean, tumble dry, or iron.
- Please follow the care instructions on the label.

6. Warranty

The manufacturer provides a 2-year warranty from the date of invoice for countries within the European Union. Statutory warranty rights remain unaffected. In the event of a warranty claim, please contact your dealer with proof of purchase.

Exclusion of warranty claims:

Damage caused by normal wear and tear, environmental influences, or improper use.
Failure to observe safety instructions.

Declaration of conformity:

Waldhausen GmbH & Co KG hereby declares that the product complies with the relevant EU directives and is CE certified.

7. Troubleshooting

Excessive temperature	Turn off the device immediately and remove it from the animal. Cool the blanket at normal room temperature.
Battery power is '0'.	Recharge the battery.
The bandage is making unusual noises	Turn off the device, check the connections, press and hold the On button.
Device does not start	Check the battery status and recharge if necessary.
Device cannot be charged	Check the USB-C port and power source

8. Disposal

Please do not dispose of the product in household waste. Dispose of it at collection points for electrical appliances.

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Spoločnosť Waldhausen GmbH & Co KG týmto vyhlasuje, že výrobok spĺňa príslušné smernice EÚ a je certifikovaný CE.
WALDHAUSEN GmbH & Co. KG